

# Instructions for Use (IFU)

## PMMA Bridge

Custom-Made Medical Device | Prescription Use Only

---

### 1. Device Description

PMMA Bridge is a custom-made prosthetic device manufactured for a single patient based on a prescription from a licensed healthcare professional. The device is supplied non-sterile.

---

### 2. Intended Use

A PMMA bridge is a custom-made prosthetic device intended to temporarily restore the form and function of one or more natural teeth for short-term use (less than 30 days). The device is Rx only, custom-made, and non-sterile.

---

### 3. Indications

Temporary restoration of the form and function of one or more natural teeth for short-term use, as prescribed by a licensed healthcare professional using standard clinical techniques.

---

### 4. Contraindications / Limitations

Not intended for long-term use.

Not intended for removable applications.

Use is limited to short-term use as defined in the Intended Use.

Not intended for use when clinical conditions do not support a temporary custom-made prosthetic restoration.

---

### 5. Prescription Statement

Rx only – For use by or on the order of a licensed healthcare professional.

---

## **6. Sterility**

Non-sterile.

---

## **7. Warnings & Precautions**

Custom-made for a single patient only.  
Intended for temporary use only (less than 30 days).  
Verify conformity with the prescription prior to use.  
Do not use if the device is damaged or altered.

---

## **8. Storage & Handling**

Handle with care to prevent damage.  
Store at room temperature in a clean, dry environment.  
Protect from contamination prior to use.

---

## **9. Manufacturer Information**

Manufacturer: Zima International, Inc. (also DBA Dandy)  
Address: N 300 W, 1320, Lehi, Utah 84043  
Country of Manufacture: United States